Wiederverwendbare Community Maske / Reusable Community Mask / Masque communautaire réutilisable -

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ENG - Composition

- Outer & inner layer: 100% polyester, fabric weight 160 g/m².
- Filter: Five-layer filter made of SSMMS nonwoven fabric (100% polypropylene), fabric weight 25g/m².

Washing and maintenance

First-time use hygiene requirements: Wash once at 60°C with all-purpose detergent. The mask can be washed at least 10 times at 60°C. Please observe the washing symbols. It is recommended to wash the mask after each use to ensure effective protection. You must dispose of your mask if its impaired in any way.

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ENG - Instructions

Before and after wearing the mask, disinfect your hands or wash them thoroughly with soap and water.

- 1.) Grasp the mask by the lower edge with your hands and pull it over your head. Do not touch the filter with your fingers. The seam of the mask should be in line with your spine (back).
- 2.) Grasp the mask by the side at the upper edge and pull well over nose and mouth. Do not touch the area of the filter.
- 3.) Tip: For a better hold and as additional protection, we recommend to tuck the mask under your ski goggles.
- 4.) When taking the mask off, grasp the upper edge (by the side, outside the filter area) and pull it over your head.

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